

# Specimen Collection & Shipping Instructions

Lumera™ Heme NGS Kit



# Specimen Collection

Specimens must be received within 48 hours of collection for optimal analysis. Keep samples at 4°C-18°C to reduce specimen degradation. Do not freeze.



## EDTA (PURPLE TOP) TUBE

with 3-6 mL **fresh bone marrow aspirate** or **peripheral blood**



### Important Information

Specimens for testing must be clearly labeled with at least **two unique identifiers** and submitted with a requisition. Adequate specimen quantity/volume must be submitted to ensure the highest quality of analysis. Ship specimens same day with frozen cold pack. Please contact us if you have any questions.



### Kit + Shipping Checklist

- Label all specimen tubes/containers with two unique identifiers.
- Include test requisition and relevant documentation/medical records.
- Package samples into the kit with frozen cold pack.
- Contact us for courier service or ship samples promptly with the provided shipping labels. Please document tracking # for your reference.