

Specimen Collection & Shipping Instructions

Lumera™ Solid Tumor Kit



Specimen Collection

This kit contains specimen containers to submit FFPE/slides for your Lumera™ Solid Tumor testing. Prepare and submit the following:



1 FFPE TUMOR BLOCK

- $\geq 20\%$ tumor content
- Include 1 H&E slide if available
- Core needle biopsies accepted

STRONGLY PREFERRED

OR



UNSTAINED SLIDES

- $\geq 20\%$ tumor content
- 4-5 micron sections
- Lumera™ Xpanded Profile: 25 slides
- Lumera™ NGS Profile: 10 slides



Important Information

Specimens for testing must be clearly labeled with at least **two unique identifiers** and submitted with a requisition. Adequate specimen quantity/volume must be submitted to ensure the highest quality of analysis. Please contact us if you have any questions.



Kit + Shipping Checklist

- Label FFPE blocks and slides/cassettes with two unique identifiers.
- Include test requisition and relevant documentation/medical records.
- FFPE blocks should be shipped at ambient temperature or with frozen cold packs during warmer summer months to prevent melting.
- Contact us for courier service or ship samples promptly with the provided shipping labels. Please document tracking # for your reference.